

Aurum

with **Manuka Honey**



Introducing the
Aurum[®] range



Innovators in **Stomacare**

Manuka Honey



The new Aurum® range of one-piece colostomy, ileostomy and urostomy pouches offer new and enhanced features that help improve the quality of lives of ostomates around the world.

One of the key new features of the Aurum® range is the introduction of medical grade Manuka Honey to our skin-friendly Hyperflex® hydrocolloid. The inclusion of Manuka Honey in our Hyperflex® hydrocolloid may help to promote healthy skin around the stoma.

Aurum Urostomy



Aurum Colostomy



Aurum Ileostomy



Aurum Stoma Cap



Manuka Honey is pure and natural made only from the nectar of the Manuka flower in New Zealand.

Throughout time, honey has been known for its many therapeutic properties. It is one of the oldest known remedies, used over thousands of years and continues to be used today. Historical research has shown this type of active Manuka Honey has characteristics that kill bacteria, suppress inflammation, and stimulate the growth of cells which may aid the healing process.





Colostomy

The new Aurum® closed range of pouches is available in mini, midi and maxi offering a comprehensive range of sizes.

The Aurum® closed pouches aim to improve the quality of life of ostomates with the following key features.

The introduction of Manuka Honey into Welland's Hyperflex® hydrocolloid aims to provide the best for your skin. The easy2peel™ tab helps to prevent sore skin by ensuring easy pouch removal. The new Dual-Carb®2 filter has an improved carbon formulation for better deodorisation and improved permeability reducing ballooning.

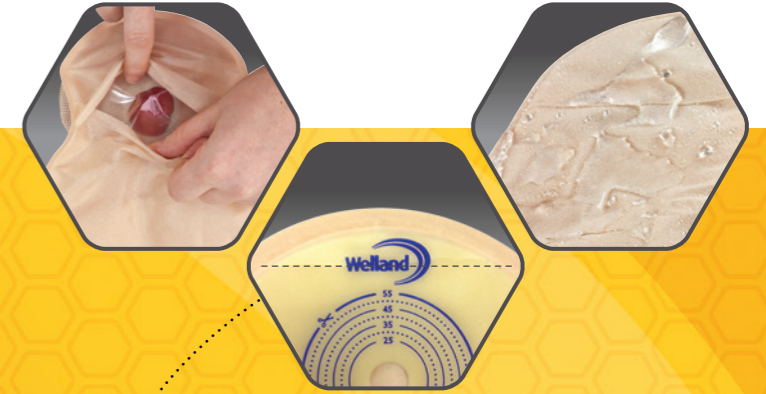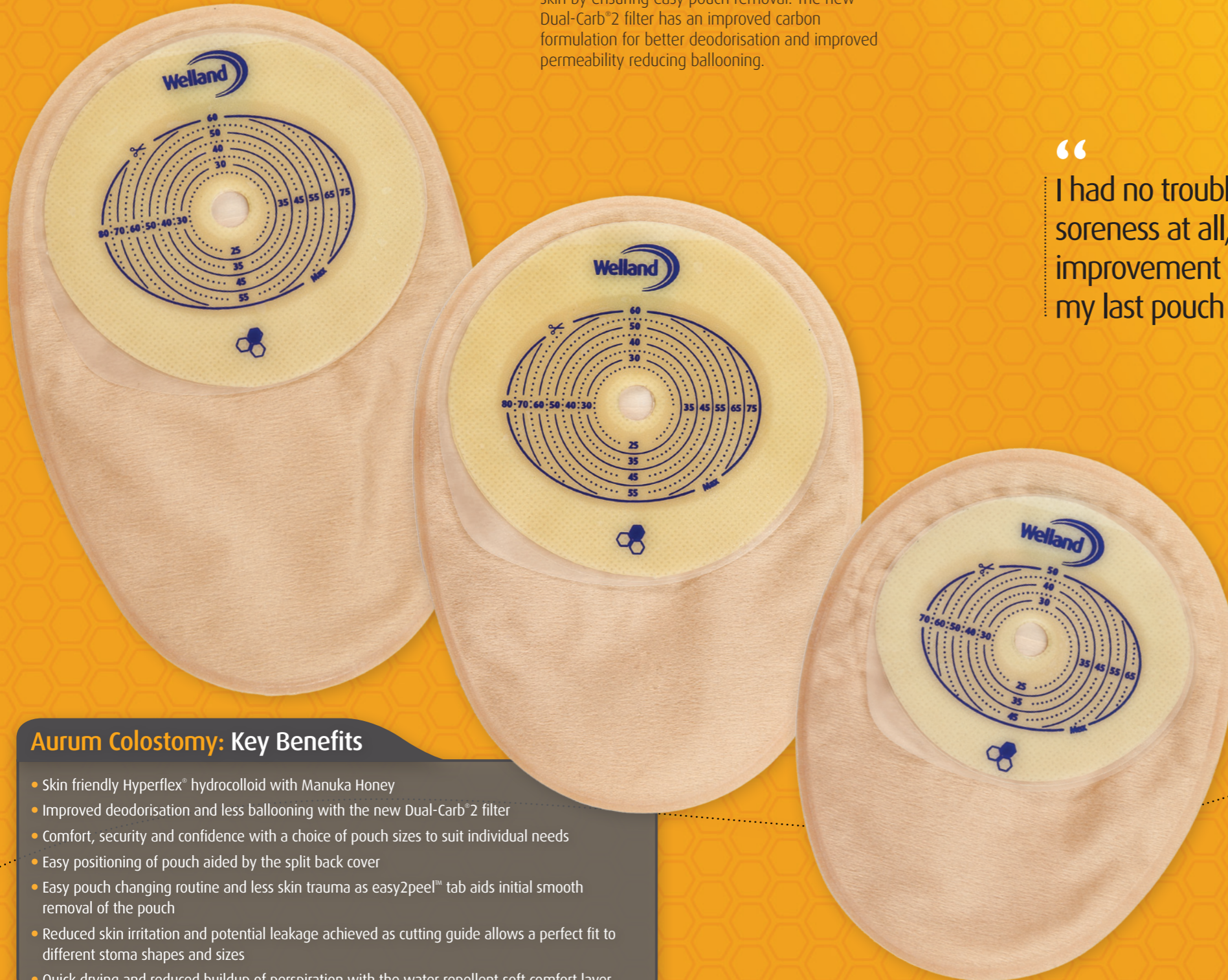

“ I had no trouble with soreness at all, a big improvement from my last pouch ”

Aurum Colostomy: Key Benefits

- Skin friendly Hyperflex® hydrocolloid with Manuka Honey
- Improved deodorisation and less ballooning with the new Dual-Carb®2 filter
- Comfort, security and confidence with a choice of pouch sizes to suit individual needs
- Easy positioning of pouch aided by the split back cover
- Easy pouch changing routine and less skin trauma as easy2peel™ tab aids initial smooth removal of the pouch
- Reduced skin irritation and potential leakage achieved as cutting guide allows a perfect fit to different stoma shapes and sizes
- Quick drying and reduced buildup of perspiration with the water repellent soft comfort layer

Aurum Closed Pouches

Product Code	Pack Size	Pouch Size	Aperture Size	Beige or Clear
XMHC513	30	Mini	13mm (50 x 70mm)	Beige
XMHC519	30	Mini	19mm	Beige
XMHC522	30	Mini	22mm	Beige
XMHC525	30	Mini	25mm	Beige
XMHC529	30	Mini	29mm	Beige
XMHC532	30	Mini	32mm	Beige
XMHC535	30	Mini	35mm	Beige
XMHC538	30	Mini	38mm	Beige
XMHC544	30	Mini	44mm	Beige
XMHC713	30	Mini	13mm (50 x 70mm)	Clear
XMHCM513	30	Midi	13mm (60 x 80mm)	Beige
XMHCM519	30	Midi	19mm	Beige
XMHCM522	30	Midi	22mm	Beige
XMHCM525	30	Midi	25mm	Beige
XMHCM529	30	Midi	29mm	Beige
XMHCM532	30	Midi	32mm	Beige
XMHCM535	30	Midi	35mm	Beige
XMHCM538	30	Midi	38mm	Beige
XMHCM544	30	Midi	44mm	Beige
XMHCM713	30	Midi	13mm (60 x 80mm)	Clear
XMHCL513	30	Maxi	13mm (60 x 80mm)	Beige
XMHCL519	30	Maxi	19mm	Beige
XMHCL522	30	Maxi	22mm	Beige
XMHCL525	30	Maxi	25mm	Beige
XMHCL529	30	Maxi	29mm	Beige
XMHCL532	30	Maxi	32mm	Beige
XMHCL535	30	Maxi	35mm	Beige
XMHCL538	30	Maxi	38mm	Beige
XMHCL544	30	Maxi	44mm	Beige
XMHCL713	30	Maxi	13mm (60 x 80mm)	Clear

Stoma Cap

Within the Aurum® range Welland Medical is pleased to introduce a stoma cap to the comprehensive product offering.

Ideal for those who irrigate or those who have regular and predictable bowel movements where the stoma is not active.

The convenience of using a stoma cap gives you the freedom to be active and provides the ultimate discretion. The stoma cap is ideal for those who irrigate to manage their colostomy or those who have regular and predictable bowel movements where the stoma is not active. The stoma cap can also be worn on a short-term basis for example during water activities, playing sports, or during intimate moments to increase discretion without compromising security.

Welland's stoma cap features the Dual-Carb® 2 filter with improved carbon formulation for better deodorisation and less ballooning, plus a water repellent soft back cover for ultimate comfort.



“The shape contours with the body making it almost undetectable under clothing”



Aurum Stoma Cap: Key Benefits

- Skin friendly Hyperflex® hydrocolloid with Manuka Honey
- Improved deodorisation and less ballooning with the new Dual-Carb®2 filter
- Reduced skin irritation and potential leakage achieved as cutting guide allows a perfect fit to different stoma shapes and sizes
- Quick drying and reduced buildup of perspiration with the water repellent soft comfort layer
- Convenient for every day activities as cap is small and discreet

Aurum Stoma Cap

Product Code	Pack Size	Pouch Size	Aperture Size	Beige or Clear
XMHSS513	30	Mini	13mm (40 x 50mm)	Beige

Ileostomy

The new Aurum® range of drainable pouches are available in mini, midi and maxi sizes. In our beige pouches the introduction of the split back cover allows easy positioning of the pouch, also new is the Dual-Carb® i filter, specifically for ileostomists.

The new Dual-Carb® i (Ileostomy) filter, features a new carbon structure which provides a larger surface area for the carbon, allowing free passage of air whilst eliminating odour.

A key feature of this filter is the PTFE membrane which repels liquid and oily substances, meaning you will experience less leakage through the filter.

“
The tab was easy
to use and emptied
much quicker ”
”



Aurum Ileostomy: Key Benefits

- Skin friendly Hyperflex® hydrocolloid with Manuka Honey
- Improved deodorisation and reduced leakage through the filter with new Dual-Carb® i filter
- Comfort, security and confidence with a choice of pouch sizes to suit individual needs
- Easy positioning of pouch aided by the split back cover
- Easy pouch changing routine and less skin trauma as easy2peel™ tab aids initial smooth removal of the pouch
- Reduced skin irritation and potential leakage achieved as cutting guide allows a perfect fit to different stoma shapes and sizes
- Easy to use and empty pouch with Welland's innovative integral closure system
- Quick drying and reduced buildup of perspiration with the water repellent soft comfort layer

Aurum Drainable Pouches

Product Code	Pack Size	Pouch Size	Aperture Size	Beige or Clear
XMHDS513	30	Mini	13mm (50 x 70mm)	Beige
XMHDS519	30	Mini	19mm	Beige
XMHDS522	30	Mini	22mm	Beige
XMHDS525	30	Mini	25mm	Beige
XMHDS529	30	Mini	29mm	Beige
XMHDS532	30	Mini	32mm	Beige
XMHDS535	30	Mini	35mm	Beige
XMHDS538	30	Mini	38mm	Beige
XMHDS544	30	Mini	44mm	Beige
XMHDS713	30	Mini	13mm (50 x 70mm)	Clear
XMHDM513	30	Midi	13mm (60 x 80mm)	Beige
XMHDM519	30	Midi	19mm	Beige
XMHDM522	30	Midi	22mm	Beige
XMHDM525	30	Midi	25mm	Beige
XMHDM529	30	Midi	29mm	Beige
XMHDM532	30	Midi	32mm	Beige
XMHDM535	30	Midi	35mm	Beige
XMHDM538	30	Midi	38mm	Beige
XMHDM544	30	Midi	44mm	Beige
XMHDM713	30	Midi	13mm (60 x 80mm)	Clear
XMHDL513	30	Maxi	13mm (60 x 80mm)	Beige
XMHDL519	30	Maxi	19mm	Beige
XMHDL522	30	Maxi	22mm	Beige
XMHDL525	30	Maxi	25mm	Beige
XMHDL529	30	Maxi	29mm	Beige
XMHDL532	30	Maxi	32mm	Beige
XMHDL535	30	Maxi	35mm	Beige
XMHDL538	30	Maxi	38mm	Beige
XMHDL544	30	Maxi	44mm	Beige
XMHDL713	30	Maxi	13mm (60 x 80mm)	Clear

Urostomy

Have the confidence to wear your pouch and continue daily activities with the new Aurum® urostomy pouch. Offering new and enhanced features including Manuka Honey in our skin friendly Hyperflex® hydrocolloid.

Now with a new improved tap pocket which is more easily tucked away to significantly reduce the profile of the pouch.

The Aurum® urostomy range is available in midi or maxi sizes with a beige split back cover or as a clear pouch.



Aurum Urostomy: Key Benefits

- Skin friendly Hyperflex® hydrocolloid with Manuka Honey
- Comfortable and discreet pouch as soft tap folds up into the tap pocket
- Comfort, security and confidence with a choice of pouch sizes to suit individual needs
- Easy positioning of pouch aided by the split back cover
- Easy pouch changing routine and less skin trauma as easy2peel™ tab aids initial smooth removal of the pouch
- Infection risk reduced as non-return valve prevents reflux of urine
- Quick drying and reduced buildup of perspiration with the water repellent soft comfort layer

“ It was great tucked out of the way, not sticking in your groin, nice and very easy to use, very comfortable ”



Aurum Urostomy Pouches

Product Code	Pack Size	Pouch Size	Aperture Size	Beige or Clear
XMHUM513	10	Midi	13mm up to 55mm	Beige
XMHUM519	10	Midi	19mm	Beige
XMHUM522	10	Midi	22mm	Beige
XMHUM525	10	Midi	25mm	Beige
XMHUM529	10	Midi	29mm	Beige
XMHUM532	10	Midi	32mm	Beige
XMHUM535	10	Midi	35mm	Beige
XMHUM538	10	Midi	38mm	Beige
XMHUM713	10	Midi	13mm up to 55mm	Clear
XMHUM719	10	Midi	19mm	Clear
XMHUM722	10	Midi	22mm	Clear
XMHUM725	10	Midi	25mm	Clear
XMHUM729	10	Midi	29mm	Clear
XMHUM732	10	Midi	32mm	Clear
XMHUM735	10	Midi	35mm	Clear
XMHUM738	10	Midi	38mm	Clear
XMHUL513	10	Maxi	13mm up to 55mm	Beige
XMHUL519	10	Maxi	19mm	Beige
XMHUL522	10	Maxi	22mm	Beige
XMHUL525	10	Maxi	25mm	Beige
XMHUL529	10	Maxi	29mm	Beige
XMHUL532	10	Maxi	32mm	Beige
XMHUL535	10	Maxi	35mm	Beige
XMHUL538	10	Maxi	38mm	Beige
XMHUL713	10	Maxi	13mm up to 55mm	Clear
XMHUL719	10	Maxi	19mm	Clear
XMHUL722	10	Maxi	22mm	Clear
XMHUL725	10	Maxi	25mm	Clear
XMHUL729	10	Maxi	29mm	Clear
XMHUL732	10	Maxi	32mm	Clear
XMHUL735	10	Maxi	35mm	Clear
XMHUL738	10	Maxi	38mm	Clear

Aurum

with Manuka Honey



Welland Medical Ltd., Hydehurst Lane, Crawley, West Sussex RH10 9AS United Kingdom
Tel: +44 (0) 1293 615455 **Fax:** +44 (0) 1293 615411 **Email:** info@wellandmedical.com (A CliniMed® Group Company)
All products marked ®, Welland® and the Welland logo™ are trade marks of CliniMed (Holdings) Limited.

www.wellandmedical.com

