

Aurum Xtra
with **Manuka Honey**



Introducing the
Aurum[®] Xtra
range



Innovators in **Stomacare**

Aurum[®]Xtra

with **Manuka Honey**



Combining two revolutionary innovations in one unique pouch.

Specifically developed to improve colostomates' quality of life; Aurum[®] Xtra is an essential solution for any colostomate whose life is restricted by the inconvenience and embarrassment of disposing of a standard closed pouch.

The Aurum[®] Xtra inner liner and flange can be flushed in domestic toilets, single slush siphonic systems and septic tanks, giving the feeling of a normal toilet experience and liberating the wearer of any anxiety they may have had before.

For those with sore and excoriated skin, Aurum[®] Xtra also has Manuka Honey added to the skin friendly Hyperflex[®] hydrocolloid flange, which may help to promote healthy skin around the stoma.



Benefits at a glance...

Skin Friendly

The Hyperflex[®] hydrocolloid flange has added Manuka Honey, which may help to promote healthy skin around the stoma

Freedom

Never have to carry a used pouch again

Flushable in domestic toilets
A time saving and convenient solution to pouch changing

Manuka Honey

This type of active Manuka Honey has characteristics that kill bacteria, suppress inflammation and stimulate the growth of cells

Flushable liner

Inner liner and flange that can be simply flushed



Features & Benefits

- The Hyperflex[®] hydrocolloid with added Manuka Honey is skin friendly and may help reduce skin irritation
- Removable flange - users can choose to separate the inner liner and flange if they wish, giving them a wider choice of how to dispose of the pouch and waste
- New Dual-Carb[®]2 filter - laser cut for optimum accuracy to ensure less ballooning and more effective management of gas and odour
- easy2peel[™] tab - supports the smooth removal of the flange, reducing skin trauma
- Super soft water repellent cover - exceptional comfort even during extended wear
- Cutting guide - allows a perfect fit for different stoma shapes and sizes, reducing skin irritation and potential leakage
- Mini, midi and maxi sizes available to suit individual needs

How to use Aurum[®] Xtra

Disposal of Aurum[®] Xtra is very simple and quick to do. Just follow the easy steps below.

Step One



Dispose of the outer pouch in the bin

Step Two



Flush the inner liner and flange down the toilet

Or separate the inner liner & flange,
the choice is yours!

Step One



Dispose of the outer pouch in the bin

Step Two



Flush the inner liner down the toilet

Step Three



Discard flange in the appropriate waste disposal bin or flush down the toilet

Your Questions Answered...

What are the benefits of having Manuka Honey in the flange?

Welland Medical's Hyperflex® hydrocolloid is formulated based on natural ingredients. Historical data supports the use of these ingredients for the application of the adhesion of ostomy pouches to the patient's skin. Manuka Honey is known for its therapeutic properties and may help to promote healthy skin around the stoma.

Manuka Honey is pure and natural, made only from the nectar of the Manuka flower in New Zealand. Throughout time, honey has been known for its many therapeutic properties. It is one of the oldest known remedies, used over thousands of years and continues to be used today. Historical research has shown this type of active Manuka Honey has characteristics that kill bacteria, suppress inflammation, and stimulate the growth of cells which may aid the healing process.

How do I know it will flush?

Welland Medical has been leading the way in flushable innovation since 1996, when the world's first flushable ostomy pouch was launched (Impact). The Welland Medical Aurum® Xtra and Flair Active® Xtra remain the only flushable ostomy pouches in the world. Aurum® Xtra has had extensive testing through flushability tests based on the UK and international standards*.



*Data held on file.

Aurum Xtra
with **Manuka Honey**

What our ostomates say

“

It flushes easily... I feel more confident when out and about.”

Mrs P.B from Hereford, UK

“

I feel this is a major improvement. It is quicker and cleaner. I say this after 19 years of use. Aurum[®] Xtra makes me feel more human.”

Mrs P.M from Bexhill, UK

“

These flushable pouches are great... they give you a more normal way of life.”

Mrs S.F from Gwent, UK

“

I find it very easy to use and much better for my skin.”

Mrs M.C from Reading, UK

“

The pouch is great! I feel more confident.”

Mrs C.D from The Netherlands



Product codes

Aurum[®] Xtra Pouches

Product Code	Pack Size	Pouch Size	Aperture Size	Beige or Clear
XMHFS519	30	Mini	19mm - 50mm	Beige
XMHFS522	30	Mini	22mm	Beige
XMHFS525	30	Mini	25mm	Beige
XMHFS529	30	Mini	29mm	Beige
XMHFS532	30	Mini	32mm	Beige
XMHFS535	30	Mini	35mm	Beige
XMHFS538	30	Mini	38mm	Beige
XMHFS544	30	Mini	44mm	Beige
XMHFM519	30	Midi	19mm - 50mm	Beige
XMHFM522	30	Midi	22mm	Beige
XMHFM525	30	Midi	25mm	Beige
XMHFM529	30	Midi	29mm	Beige
XMHFM532	30	Midi	32mm	Beige
XMHFM535	30	Midi	35mm	Beige
XMHFM538	30	Midi	38mm	Beige
XMHFM544	30	Midi	44mm	Beige
XMHFL519	30	Maxi	19mm - 50mm	Beige
XMHFL522	30	Maxi	22mm	Beige
XMHFL525	30	Maxi	25mm	Beige
XMHFL529	30	Maxi	29mm	Beige
XMHFL532	30	Maxi	32mm	Beige
XMHFL535	30	Maxi	35mm	Beige
XMHFL538	30	Maxi	38mm	Beige
XMHFL544	30	Maxi	44mm	Beige

Aurum^{Xtra} with Manuka Honey



Welland Medical Ltd., Hydehurst Lane, Crawley, West Sussex RH10 9AS United Kingdom
Tel: +44 (0) 1293 615455 **Fax:** +44 (0) 1293 615411 **Email:** info@wellandmedical.com (A CliniMed® Group Company)
All products marked ®, Welland® and the Welland logo™ are trade marks of CliniMed (Holdings) Limited.

www.wellandmedical.com

