


How would you describe the location of the problematic skin around your stoma?

Contact



USING THE DESCRIPTIONS BELOW:


- Dry
- Good/dry
- Dry/red
- Another specific issue
- Combination of above
- Sore
- Broken
- Weeping
- Itchy
- Red
- Bleeding

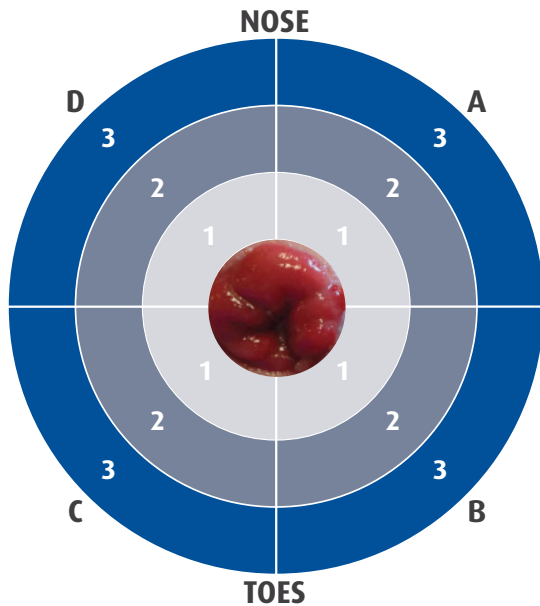

Blank area for user input.

Welland Medical Ltd., Hydehurst Lane, Crawley, West Sussex RH10 9AS United Kingdom
Tel: +44 (0) 1293 615455 **Fax:** +44 (0) 1293 615411 **Email:** info@wellandmedical.com
 ©Welland Medical 2020 • Welland Medical Ltd. is a CliniMed® Group Company
 Welland® and the Welland logo™ are trademarks of CliniMed (Holdings) Ltd.

www.wellandmedical.com

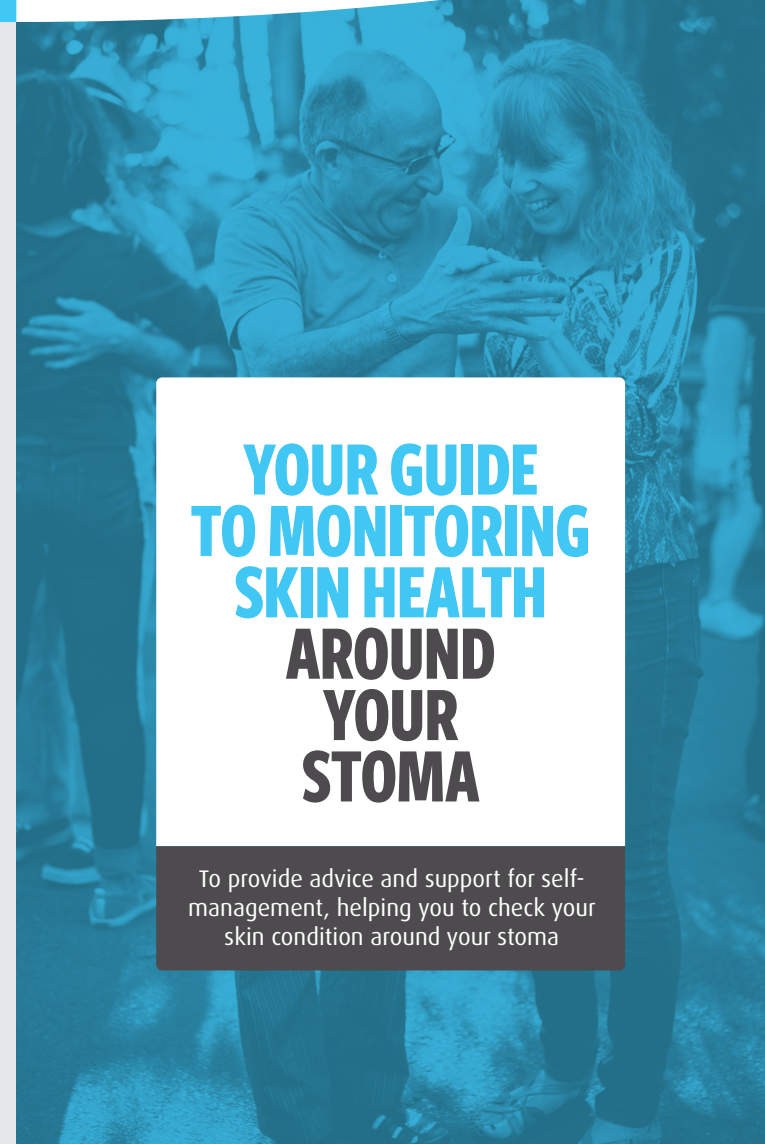

**YOUR GUIDE
 TO MONITORING
 SKIN HEALTH
 AROUND
 YOUR
 STOMA**

To provide advice and support for self-management, helping you to check your skin condition around your stoma

Along with the visual guide above, please describe the affected skin around your stoma eg. area A1 red and itchy


© SecuriCare (Medical) Ltd. (2020) All rights reserved.
 No part of this publication can be reproduced, stored in a retrieval system, or transmitted in any form or by any mean, electronic, mechanical, photocopying, recording or otherwise without the prior written permission of the author. Printed with the kind permission of SecuriCare (Medical) Ltd. PID 7226 and PID 8880

Contact your Stoma Therapy Nurse or distributor if you have any problems or concerns

THIS SELF-CARE PATHWAY IS DESIGNED TO HELP YOU CARE FOR THE SKIN AROUND YOUR STOMA



1 Please look for signs of faecal/ urinary leakage (dependent upon your stoma type) on your skin during pouch removal and check the removed flange to see if there is evidence of any leakage

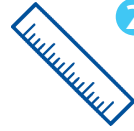

2 Please re-measure your stoma template to ensure a correct fit, to prevent exposed skin around the stoma edge



3 Please ensure that your skin is clean and dry before reapplying a crease-free stoma flange to your skin

If you are finding that you are unable to leave your pouch on for your normal length of time due to your skin condition, or your skin appearance is more severe than shown below and/or if you have any other concerns please contact your Stoma Therapy Nurse for advice and reassurance.



CHANGING YOUR POUCH

- 1 Wash your hands. If you are not near running water you can use an antibacterial hand cleanser.
- 2 Supporting the skin with one hand, gently and slowly, ease the pouch off.
- 3 Use plain warm water and dry wipe to gently clean around the stoma. Make sure you don't rub.
- 4 Dry the skin thoroughly with a dry wipe. Pat gently, once again taking care not to rub.

NORMAL/GOOD

The appearance of the skin around the stoma is the same as the rest of your skin on your abdomen.



ADVICE

Maintain usual stoma care change routine and continue to monitor.

MILD

Dry, good/dry, red, dry/red, itchy.



ADVICE

Consider a silicone medical adhesive remover (wipe/spray) to prevent skin stripping following pouch removal. Consider a silicone barrier film wipe to protect the skin around the stoma at each pouch change.

MODERATE

Red, itchy, sore or mixed/combination.



ADVICE

WET surrounding skin: consider use of stoma powder to the wet area.
DRY surrounding skin: wipe around the area with a silicone barrier film wipe (to add protection).

SEVERE

Red, itchy/sore/broken/weeping/bleeding or mixed combination/or another specific issue.



ADVICE

Contact your Stoma Care Nurse immediately.

If improvement noted,
continue to use and monitor.

If no improvement seen after 3 days,
please contact your Stoma Therapy Nurse.