

Hydrocolloid
flange extenders for
extra adhesion

Hydro  Frame®

CliniMed®

EC REP Advena Limited,
Tower Business Centre,
2nd Flr., Tower Street,
Swatar, BKR 4013 Malta

PPR-INS-B33 11



Welland Medical Ltd., Hydehurst Lane, Crawley, West Sussex RH10 9AS, United Kingdom

Welland Medical Ltd. is a CliniMed Group Company. Welland® the Welland logo™ and HydroFrame® are trademarks of CliniMed (Holdings) Ltd.

HydroFrame®

Instructions for use

HydroFrame® can be used before or after your appliance has been placed on your body.

Ensure the skin is dry.



1. Peel away middle piece of backing paper.
2. Holding by the tabs, position where appropriate.
3. Remove the remaining backing from the tabs and smooth HydroFrame® securely into place.

Remove with your appliance in the usual way.

Flange extenders – to provide extra security/comfort for an ostomy appliance.

Caution: For use on intact skin only. Reuse of any single use device may compromise safety and performance. Discontinue use if irritation occurs.

In the event of a serious incident, seek medical advice before contacting the manufacturer, Welland Medical Ltd., and your local Competent Authority via your distributor or importer.

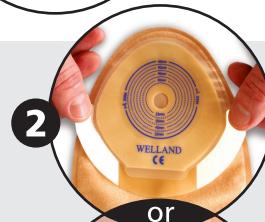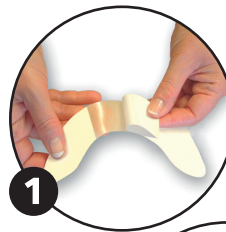

or



Careline
0800 585125

email: info@clinimed.co.uk

PID 8588 • 20-08-2020