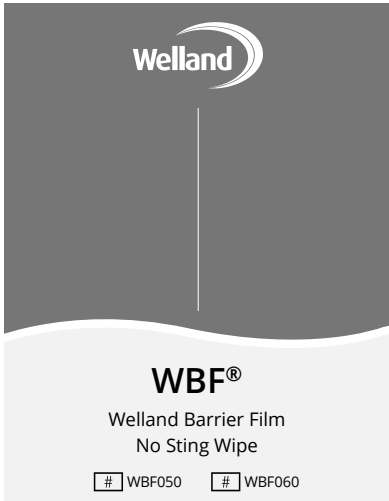
Welland Medical Ltd.,
Hydehurst Lane, Crawley,
West Sussex, RH10 9AS
United Kingdom
Tel: +44 (0) 1293 615455
www.wellandmedical.com

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EN

For Ostomates

No Sting WBF® wipes: Protects peristomal skin from contact with stoma output and osomy adhesive related skin stripping.

Caution: For use on intact peristomal skin only. Use only as directed. Avoid delicate or sensitive areas of skin. In doubt, patch test a small area prior to use. Discontinue use if irritation occurs. Reuse of any single use device may compromise safety and performance. This product may not be suitable for use on babies under 30 days old. Do not inhale. Use in well-ventilated areas. Use immediately after opening sachet.

Danger

Highly flammable liquid and vapor.

Maintaining healthy peristomal skin is vital to prevent complications when living with a stoma. Regularly check and maintain the health of the skin around your stoma, comparing it with the skin on the rest of your abdomen. If you notice any signs of peristomal skin complications such as colour change, irritation, or changes in texture, it's essential to consult a healthcare professional promptly. Untreated complications can have serious consequences, potentially leading to systemic health problems. Proactive peristomal skin care and early intervention are your best defence against complications, significantly improving your overall well-being.

In the event of a serious incident, seek medical advice before contacting the manufacturer, Welland Medical Ltd., and your local Competent Authority via your distributor or importer.

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trade marks of ClinMed (Holdings) Ltd.