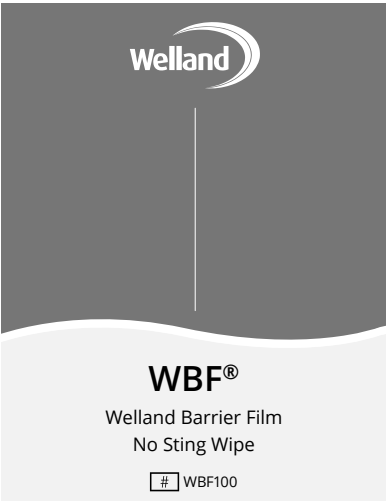
Welland Medical Ltd.,
Hydehurst Lane, Crawley,
West Sussex, RH10 9AS
United Kingdom
Tel: +44 (0) 1293 615455
www.wellandmedical.com

Welland® and the Welland logo™ are
trade marks of CliniMed (Holdings) Ltd.
PPR-INS-WF1 08 • PID 1622
15/01/2025





EU Medical Device Regulation compliant / Conforme à la
réglementation européenne sur les dispositifs médicaux
/ Conforme al Regolamento UE sui dispositivi medici /
Entspricht der EU-Medizinprodukteverordnung /
Voldoet aan EU-verordening voor medische hulpmiddelen
/ Overholder EU's regler om medicinsk udstyr /
Följer EU:s direktiv om medicintekniska produkter

Keep dry / Garder au sec / Tenere asciutto /
Trocken halten / Droog bewaren /
Opbevares tørt / Förvaras torrt

Consult instructions for use / Consulter le mode d'emploi
/ Consultare le istruzioni per l'uso / Gebrauchsanweisung
beachten / Raadpleeg de gebruiksaanwijzing /
Se brugsanvisningen / Läs bruksanvisningen

Do not re-use / Ne pas réutiliser / Non riutilizzare /
Nicht wiederverwenden / Niet hergebruiken /
Må ikke genbruges / Får inte återanvändas

Recyclable (check local systems) / Recyclable
(vérifiez les systèmes locaux) / Riciclabile
(controllare le disposizioni locali) / Recyclebar
(bitte örtliche Systeme prüfen) / Recyclebaar (controleer
lokale systemen) / Genanvendelige (kontroller lokale
systemer) / Återvinningsbar (kontrollera lokala system)

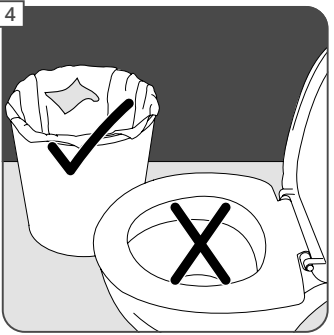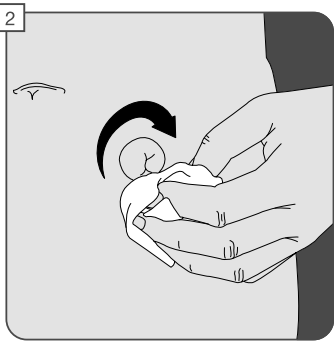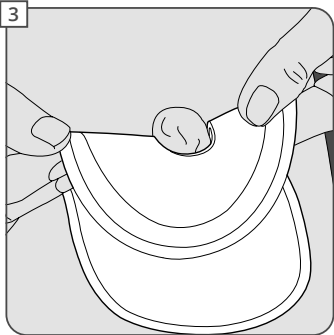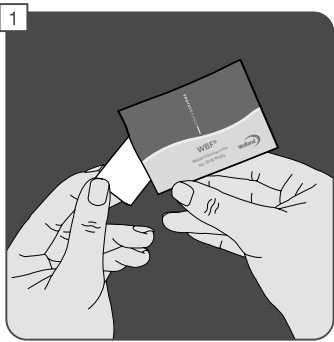Omnigon Pty Ltd.
Suite 2, Level 2, 582 Swan Street
Burnley VIC 3121 Australia
Tel: AU 1800 819 274 /
NZ 0800 440 027

EU Authorised Representative /
Représentant autorisé de l'UE / Rappresentante
Autorizzato nella UE / EU-Bevollmächtigter / Gemachtigt
vertegenwoordiger EU / EU-autoriseret repræsentant /
Auktoriserad representant i Europeiska unionen

Swiss Authorised Representative / Représentant autorisé
en Suisse / Rappresentante Autorizzato in Svizzera /
Bevollmächtigter in der Schweiz / Zwitterse geautoriseerde
vertegenwoordiger / Schweizisk autoriseret repræsentant /
Auktoriserad representant i Schweiz

Do not flush / Ne pas jeter dans les toilettes / Non gettare
nella toilette / Nicht herunterspülen / Niet doorspoelen /
Må ikke skylles ud i toalettet / Får inte spolas ner i toaletten

Manufacturer / Fabricant / Produttore / Hersteller /
Fabrikant / Producent / Tillverkare

Date of manufacture / Date de fabrication /
Data di fabbricazione / Datum der Herstellung /
Productiedatum / Fremstillingsdato / Tillverkningsdatum

Batch code / Code de lot / Codice lotto /
Chargennummer / Batchnummer / Batchkode / Batch

EN

For Ostomates

No Sting WBF® wipes: Protects peristomal skin from contact with stoma output and
ostomy adhesive related skin stripping.

Caution: For use on intact peristomal skin only. Use only as directed. Avoid delicate
or sensitive areas of skin. If in doubt, patch test a small area prior to use. Discontinue
use if irritation occurs. Reuse of any single use device may compromise safety and
performance. This product may not be suitable for use on babies under 30 days old. Do
not inhale. Use in well-ventilated areas. Use immediately after opening sachet.

Danger

Highly flammable liquid and vapour.

Maintaining healthy peristomal skin is vital to prevent complications when living with
a stoma. Regularly check and maintain the health of the skin around your stoma,
comparing it with the skin on the rest of your abdomen. If you notice any signs of
peristomal skin complications such as colour change, irritation, or changes in texture,
it's essential to consult a healthcare professional promptly. Untreated complications
can have serious consequences, potentially leading to systemic health problems.
Proactive peristomal skin care and early intervention are your best defence against
complications, significantly improving your overall well-being.

In the event of a serious incident, seek medical advice before contacting the
manufacturer, Welland Medical Ltd., and your local Competent Authority via your
distributor or importer.

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de fabrikant, Welland Medical Ltd., en uw plaatselijke bevoegde autoriteit via uw
tillverkaren, Welland Medical Ltd., och din lokala reglerande myndighet via din