



Aurum[®] Plus

The Positive Choice





A range of stoma bags designed around more than just function – it's about feeling at ease, protected and confident every single day.

Aurum[®] Plus 



The
TRUSTED
choice

The plus shaped flange is purposefully designed to adapt to natural body contours with precision, for a personalised and secure fit.



The
STYLISH
choice

To seamlessly match to activities, lifestyles and clothing preferences, enhancing discretion and empowering individuals.

There's more than one plus...



The
SKIN-FRIENDLY
choice

Our Manuka honey infused hydrocollid is dermatologically accredited, affirming it is safe to use and kind to skin.



The
CLINICAL
choice

The perfect choice for Every Body – A wide selection of bags to meet the unique needs of different stoma types and body profiles.

There's more than one Plus

Aurum® Plus Soft Convex is ideal for those with a dip or moat around their stoma, or where the stoma is flush, recessed or retracted and needs help to protrude.




6mm convexity depth across the full range.

The soft belt loops on Aurum® Plus Soft Convex which are hidden behind the flange, increase tension for added security without causing discomfort by digging into the skin.

Available in a choice of colours: black and sand, to suit individual preferences, activities and lifestyle choices.

Thicker hydrocolloid on urostomy products have a much higher absorption capacity, reducing maceration and moisture associated skin damage (MASD).

Features our dermatologically accredited Manuka honey infused hydrocolloid.

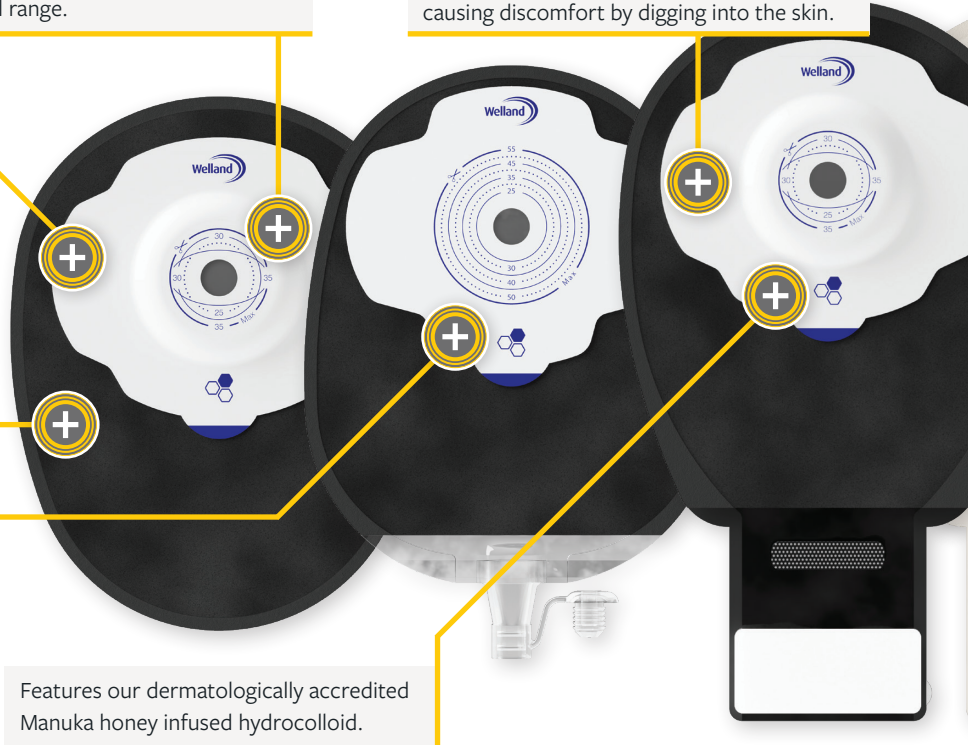

The flat flange on Aurum® Plus is ideal for stomas which protrude and have an even, flat peristomal surface.



Aurum® Plus Soft Convex is available in four tension location sizes 25mm, 35mm, 48mm, and 60mm* to allow users to effectively apply tension around their peristomal skin.

The plus shaped flange is purposefully designed to reduce the risk of leak channels forming.

To address common leak-prone areas between the 3 o'clock and 9 o'clock positions, our wide flange offers optimal adhesion for those vulnerable areas prone to leakage ensuring a secure fit across the problematic midline bend of the abdomen.

Available in a 13mm starter hole and a wide variety of pre-cuts.

Available in closed, drainable and urostomy in a wide range of bag sizes to suit a variety of body profiles.

*Closed and drainable bags only


There's more than one Plus

Water repellent bag material is soft against the skin and dries quickly saving time waiting for the bag to dry.


The tap pocket keeps the tap securely tucked away and secure, enhancing discretion.

The split back cover allows you to visually confirm the bag is positioned well and monitor the stoma and output, enhancing confidence in daily stoma care routines.

Emptying the drainable bag is easy with the large, easy to use pull tab.

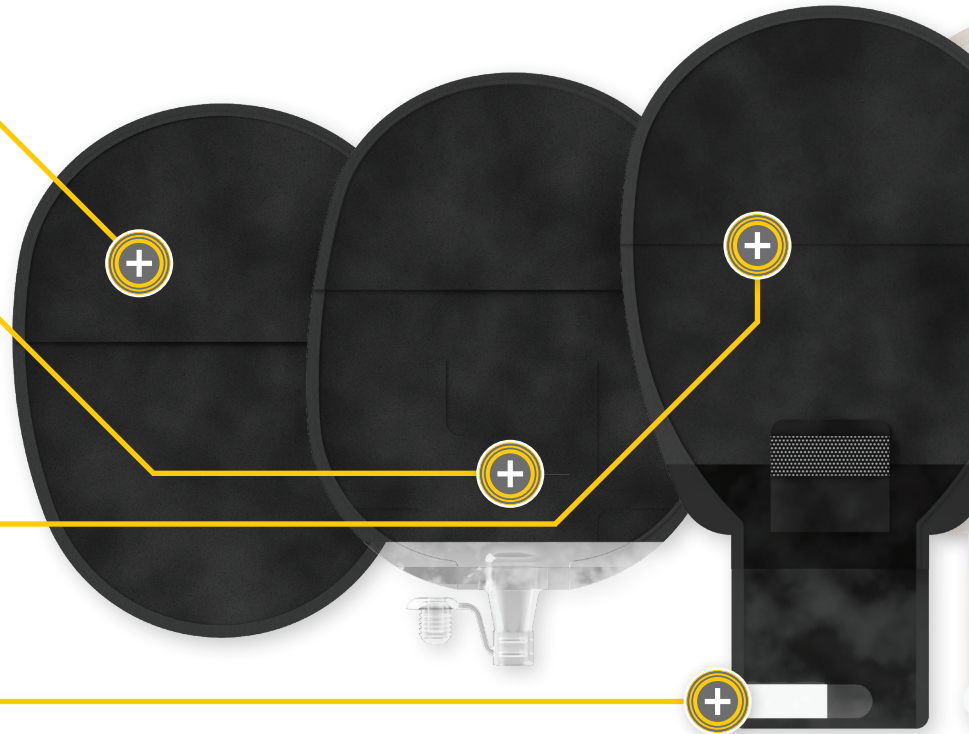

Aurum[®] Plus 

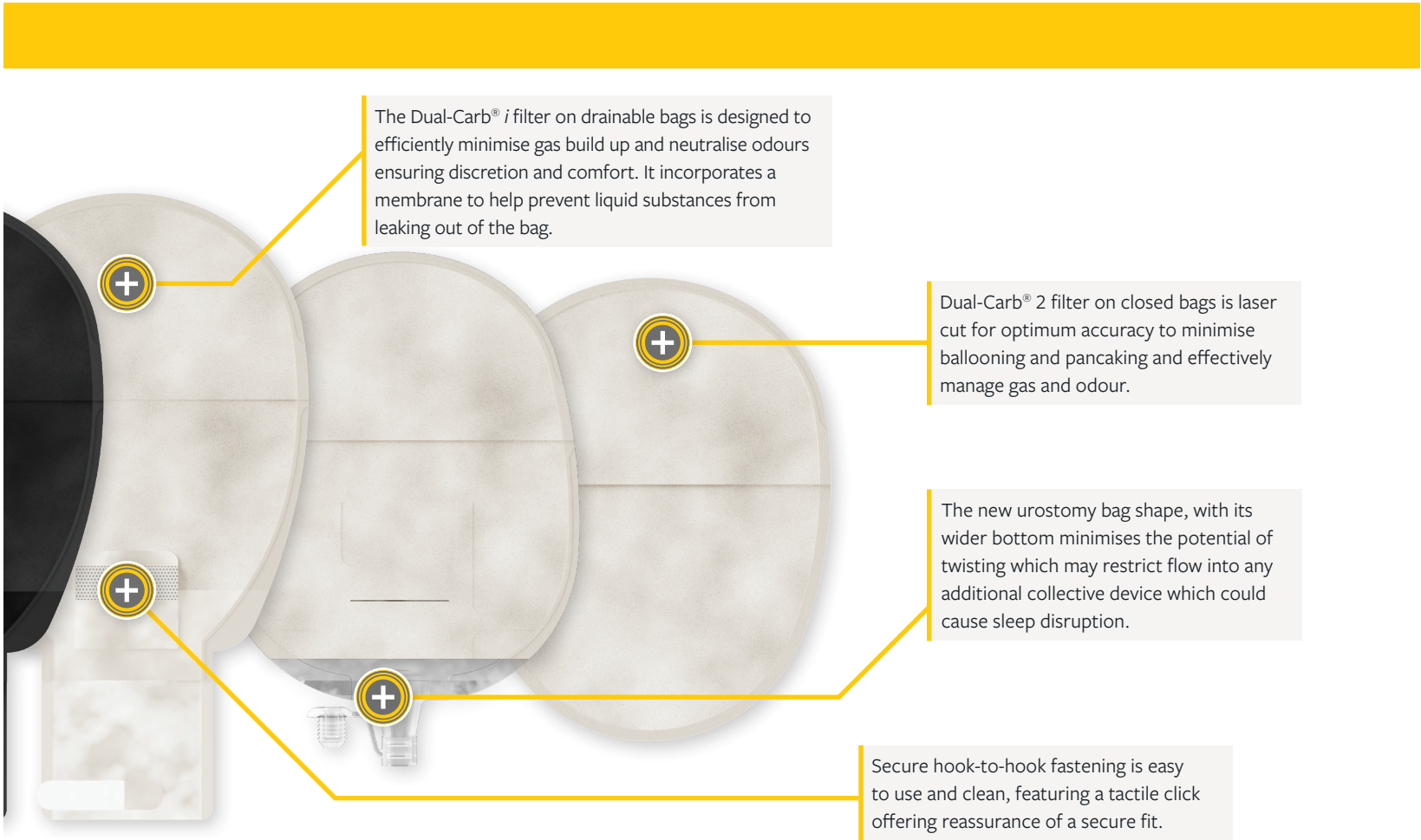

The Dual-Carb® *i* filter on drainable bags is designed to efficiently minimise gas build up and neutralise odours ensuring discretion and comfort. It incorporates a membrane to help prevent liquid substances from leaking out of the bag.

Dual-Carb® 2 filter on closed bags is laser cut for optimum accuracy to minimise ballooning and pancaking and effectively manage gas and odour.

The new urostomy bag shape, with its wider bottom minimises the potential of twisting which may restrict flow into any additional collective device which could cause sleep disruption.

Secure hook-to-hook fastening is easy to use and clean, featuring a tactile click offering reassurance of a secure fit.

The TRUSTED choice



Our plus shaped flange adapts to natural body contours with precision. It provides a trusted, personalised fit through optimal adhesion, creating a secure seal and minimising leakage.

Trusted Fit, Every Time.

Trusted worldwide for its personalised and secure fit, our purposefully designed plus shape flange conforms to the body's natural contours with precision, giving our users lasting confidence without compromise.

Our unique flange provides a trusted, personalised fit across the natural bend line, creating a secure seal and minimising leaks.



The
SKIN-
FRIENDLY
choice



Our dermatologically accredited Manuka honey hydrocolliod increases comfort, security and confidence in patients with peristomal skin complications.

We Care About Skin. It's That Simple.

Skin health is one of the most important things in stoma care, without good skin health, a simple stoma soon becomes a complex one to manage.

Our dermatologically accredited hydrocolloid is globally recognised by, skin health accreditation body, The Skin Health Alliance (SHA). This accreditation demonstrates that our products meet or exceed strict scientific standards for skin safety and efficiency, meaning you can use and recommend our products with confidence.

The integration of natural food-grade ingredients with our hydrocolloid blend, free from tackifiers, is reported by users worldwide to offer greater comfort and peace of mind during wear time. Fewer ingredients lowers the risk of allergies and irritation and ensures a comfortable experience for all users.



Dermatologically Accredited

White and Evans (2019) demonstrated that a Manuka honey infused hydrocolloid increased comfort, security and confidence in patients.*

They reported that 94% of patients saw an improvement in their peristomal skin within 7 days.

*N= 58 patients.

The STYLISH choice



Our bags are available in black and sand to enhance discretion and empower individuals by offering a choice which can match the users' activities, lifestyles and clothing preferences.

Your Bag. Your Choice.

Coloured bags play a crucial role in empowering users to feel confident in their daily lives.

Beyond functionality, these bags allow individuals to coordinate with their personal style and lifestyle choices, helping to ease self-consciousness and promote a positive quality of life.

Our sand bag is a refreshing take on our traditional stoma bag, offering a sense of reassurance and familiarity while embracing a modern touch they can feel confident in and style with lighter clothing. While our sleek black bag offers a contemporary twist on the stoma bag, giving users the choice to make a bold statement or remain discreet by blending in with darker clothing.



The CLINICAL choice



The perfect choice for Every Body - A wide selection of bags and tension location sizes to meet the unique needs of different stoma types and body profiles.

Something For Every Body.

Aurum® Plus has a wide selection of bags available in multiple sizes, pre-cut options, and a choice of colours. This variety allows users to choose a bag that perfectly matches their individual needs, activities and lifestyles.

A Range Of Tension Location Sizes For The Perfect Fit.

To prevent leakage problems personalise the tension levels on the peristomal area with one of four tension location sizes for a custom and comfortable fit.

Available in 25mm, 35mm, 48mm and 60mm*, to apply tension either around the base of the stoma to help it protrude, or away from the stoma to flatten the peristomal skin.

Enhanced Protrusion and Stability

Choose a smaller tension location size to concentrate tension centrally around the stoma, providing heightened stability at the base helping the stoma to protrude.



Flatten and Smooth Peristomal Skin

Choose a larger tension location size to direct tension peripherally outward and effectively flatten out creases to create a smoother surface away from the stoma.

*Closed and drainable bags only

“

I like using the Aurum® Plus black bag because it makes me feel confident and more sexy in my own skin.

Having a choice of colour options for my bag is very empowering.”





“

I never had any irritation while using the Manuka honey bag, it keeps my skin free from redness or irritation.

It is very easy to use, and I know my skin is in good condition with this product.”

Jess Skipper, Australia
uses Aurum[®] Plus drainable



KEY



Black colour bag



Sand colour bag



Clear bag



Aurum[®] Plus

Aurum[®] Plus Soft Convex Closed

Product Code	Pack Size	Bag Size	Tension Location	Depth	Aperture Size
X1C1N126113	10	Midi	25mm	6mm	13mm Starter Hole
X1C1N126120	10	Midi	25mm	6mm	20mm
X1C1N126125	10	Midi	25mm	6mm	25mm
X1C1N226113	10	Midi	25mm	6mm	13mm Starter Hole
X1C1N226120	10	Midi	25mm	6mm	20mm
X1C1N226125	10	Midi	25mm	6mm	25mm
X1C1N136113	10	Maxi	25mm	6mm	13mm Starter Hole
X1C1N136120	10	Maxi	25mm	6mm	20mm
X1C1N136125	10	Maxi	25mm	6mm	25mm
X1C1N236113	10	Maxi	25mm	6mm	13mm Starter Hole
X1C1N236120	10	Maxi	25mm	6mm	20mm
X1C1N236125	10	Maxi	25mm	6mm	25mm
X1C1N336113	10	Maxi	25mm	6mm	13mm Starter Hole
X1C1N126213	10	Midi	35mm	6mm	13mm Starter Hole
X1C1N126220	10	Midi	35mm	6mm	20mm
X1C1N126225	10	Midi	35mm	6mm	25mm
X1C1N126229	10	Midi	35mm	6mm	29mm
X1C1N126232	10	Midi	35mm	6mm	32mm
X1C1N126235	10	Midi	35mm	6mm	35mm
X1C1N226213	10	Midi	35mm	6mm	13mm Starter Hole
X1C1N226220	10	Midi	35mm	6mm	20mm
X1C1N226225	10	Midi	35mm	6mm	25mm
X1C1N226229	10	Midi	35mm	6mm	29mm
X1C1N226232	10	Midi	35mm	6mm	32mm
X1C1N226235	10	Midi	35mm	6mm	35mm
X1C1N326213	10	Midi	35mm	6mm	13mm Starter Hole
X1C1N136213	10	Maxi	35mm	6mm	13mm Starter Hole
X1C1N136220	10	Maxi	35mm	6mm	20mm
X1C1N136225	10	Maxi	35mm	6mm	25mm
X1C1N136229	10	Maxi	35mm	6mm	29mm
X1C1N136232	10	Maxi	35mm	6mm	32mm
X1C1N136235	10	Maxi	35mm	6mm	35mm
X1C1N236213	10	Maxi	35mm	6mm	13mm Starter Hole

Aurum[®] Plus Soft Convex Closed

Product Code	Pack Size	Bag Size	Tension Location	Depth	Aperture Size
X1C1N236220	10	Maxi	35mm	6mm	20mm
X1C1N236225	10	Maxi	35mm	6mm	25mm
X1C1N236229	10	Maxi	35mm	6mm	29mm
X1C1N236232	10	Maxi	35mm	6mm	32mm
X1C1N236235	10	Maxi	35mm	6mm	35mm
X1C1N336213	10	Maxi	35mm	6mm	13mm Starter Hole
X1C1N126313	10	Midi	48mm	6mm	13mm Starter Hole
X1C1N126332	10	Midi	48mm	6mm	32mm
X1C1N126335	10	Midi	48mm	6mm	35mm
X1C1N126338	10	Midi	48mm	6mm	38mm
X1C1N126344	10	Midi	48mm	6mm	44mm
X1C1N226313	10	Midi	48mm	6mm	13mm Starter Hole
X1C1N226332	10	Midi	48mm	6mm	32mm
X1C1N226335	10	Midi	48mm	6mm	35mm
X1C1N226338	10	Midi	48mm	6mm	38mm
X1C1N226344	10	Midi	48mm	6mm	44mm
X1C1N326313	10	Midi	48mm	6mm	13mm Starter Hole
X1C1N136313	10	Maxi	48mm	6mm	13mm Starter Hole
X1C1N136332	10	Maxi	48mm	6mm	32mm
X1C1N136335	10	Maxi	48mm	6mm	35mm
X1C1N136338	10	Maxi	48mm	6mm	38mm
X1C1N136344	10	Maxi	48mm	6mm	44mm
X1C1N236313	10	Maxi	48mm	6mm	13mm Starter Hole
X1C1N236332	10	Maxi	48mm	6mm	32mm
X1C1N236335	10	Maxi	48mm	6mm	35mm
X1C1N236338	10	Maxi	48mm	6mm	38mm
X1C1N236344	10	Maxi	48mm	6mm	44mm
X1C1N336313	10	Maxi	48mm	6mm	13mm Starter Hole
X1C1N146413	10	Midi Plus	60mm	6mm	13mm Starter Hole
X1C1N246413	10	Midi Plus	60mm	6mm	13mm Starter Hole
X1C1N156413	10	Maxi Plus	60mm	6mm	13mm Starter Hole
X1C1N256413	10	Maxi Plus	60mm	6mm	13mm Starter Hole
X1C1N356413	10	Maxi Plus	60mm	6mm	13mm Starter Hole



Aurum® Plus Soft Convex Drainable




Product Code	Pack Size	Bag Size	Tension Location	Depth	Aperture Size
X1D1N126113	10	Midi	25mm	6mm	13mm Starter Hole
X1D1N126120	10	Midi	25mm	6mm	20mm
X1D1N126125	10	Midi	25mm	6mm	25mm
X1D1N226113	10	Midi	25mm	6mm	13mm Starter Hole
X1D1N226120	10	Midi	25mm	6mm	20mm
X1D1N226125	10	Midi	25mm	6mm	25mm
X1D1N136113	10	Maxi	25mm	6mm	13mm Starter Hole
X1D1N136120	10	Maxi	25mm	6mm	20mm
X1D1N136125	10	Maxi	25mm	6mm	25mm
X1D1N236113	10	Maxi	25mm	6mm	13mm Starter Hole
X1D1N236120	10	Maxi	25mm	6mm	20mm
X1D1N236125	10	Maxi	25mm	6mm	25mm
X1D1N336113	10	Maxi	25mm	6mm	13mm Starter Hole
X1D1N126213	10	Midi	35mm	6mm	13mm Starter Hole
X1D1N126220	10	Midi	35mm	6mm	20mm
X1D1N126225	10	Midi	35mm	6mm	25mm
X1D1N126229	10	Midi	35mm	6mm	29mm
X1D1N126232	10	Midi	35mm	6mm	32mm
X1D1N126235	10	Midi	35mm	6mm	35mm
X1D1N226213	10	Midi	35mm	6mm	13mm Starter Hole
X1D1N226220	10	Midi	35mm	6mm	20mm
X1D1N226225	10	Midi	35mm	6mm	25mm
X1D1N226229	10	Midi	35mm	6mm	29mm
X1D1N226232	10	Midi	35mm	6mm	32mm
X1D1N226235	10	Midi	35mm	6mm	35mm
X1D1N326213	10	Midi	35mm	6mm	13mm Starter Hole
X1D1N136213	10	Maxi	35mm	6mm	13mm Starter Hole
X1D1N136220	10	Maxi	35mm	6mm	20mm
X1D1N136225	10	Maxi	35mm	6mm	25mm
X1D1N136229	10	Maxi	35mm	6mm	29mm
X1D1N136232	10	Maxi	35mm	6mm	32mm
X1D1N136235	10	Maxi	35mm	6mm	35mm
X1D1N236213	10	Maxi	35mm	6mm	13mm Starter Hole

Aurum® Plus Soft Convex Drainable

Product Code	Pack Size	Bag Size	Tension Location	Depth	Aperture Size
X1D1N236220	10	Maxi	35mm	6mm	20mm
X1D1N236225	10	Maxi	35mm	6mm	25mm
X1D1N236229	10	Maxi	35mm	6mm	29mm
X1D1N236232	10	Maxi	35mm	6mm	32mm
X1D1N236235	10	Maxi	35mm	6mm	35mm
X1D1N336213	10	Maxi	35mm	6mm	13mm Starter Hole
X1D1N126313	10	Midi	48mm	6mm	13mm Starter Hole
X1D1N126332	10	Midi	48mm	6mm	32mm
X1D1N126335	10	Midi	48mm	6mm	35mm
X1D1N126338	10	Midi	48mm	6mm	38mm
X1D1N126344	10	Midi	48mm	6mm	44mm
X1D1N226313	10	Midi	48mm	6mm	13mm Starter Hole
X1D1N226332	10	Midi	48mm	6mm	32mm
X1D1N226335	10	Midi	48mm	6mm	35mm
X1D1N226338	10	Midi	48mm	6mm	38mm
X1D1N226344	10	Midi	48mm	6mm	44mm
X1D1N326313	10	Midi	48mm	6mm	13mm Starter Hole
X1D1N136313	10	Maxi	48mm	6mm	13mm Starter Hole
X1D1N136332	10	Maxi	48mm	6mm	32mm
X1D1N136335	10	Maxi	48mm	6mm	35mm
X1D1N136338	10	Maxi	48mm	6mm	38mm
X1D1N136344	10	Maxi	48mm	6mm	44mm
X1D1N236313	10	Maxi	48mm	6mm	13mm Starter Hole
X1D1N236332	10	Maxi	48mm	6mm	32mm
X1D1N236335	10	Maxi	48mm	6mm	35mm
X1D1N236338	10	Maxi	48mm	6mm	38mm
X1D1N236344	10	Maxi	48mm	6mm	44mm
X1D1N336313	10	Maxi	48mm	6mm	13mm Starter Hole
X1D1N146413	10	Midi Plus	60mm	6mm	13mm Starter Hole
X1D1N246413	10	Midi Plus	60mm	6mm	13mm Starter Hole
X1D1N156413	10	Maxi Plus	60mm	6mm	13mm Starter Hole
X1D1N256413	10	Maxi Plus	60mm	6mm	13mm Starter Hole
X1D1N356413	10	Maxi Plus	60mm	6mm	13mm Starter Hole



KEY

-  Black colour bag
-  Sand colour bag
-  Clear bag



Aurum® Plus Soft Convex Urostomy

Product Code	Pack Size	Bag Size	Tension Location	Depth	Aperture Size
X1U1R126113	10	Midi	25mm	6mm	13mm Starter Hole
X1U1R126122	10	Midi	25mm	6mm	22mm
X1U1R126125	10	Midi	25mm	6mm	25mm
X1U1R226113	10	Midi	25mm	6mm	13mm Starter Hole
X1U1R226122	10	Midi	25mm	6mm	22mm
X1U1R226125	10	Midi	25mm	6mm	25mm
X1U1R136113	10	Maxi	25mm	6mm	13mm Starter Hole
X1U1R136122	10	Maxi	25mm	6mm	22mm
X1U1R136125	10	Maxi	25mm	6mm	25mm
X1U1R236113	10	Maxi	25mm	6mm	13mm Starter Hole
X1U1R236122	10	Maxi	25mm	6mm	22mm
X1U1R236125	10	Maxi	25mm	6mm	25mm
X1U1R336113	10	Maxi	25mm	6mm	13mm Starter Hole
X1U1R126213	10	Midi	35mm	6mm	13mm Starter Hole
X1U1R126222	10	Midi	35mm	6mm	22mm
X1U1R126225	10	Midi	35mm	6mm	25mm
X1U1R126229	10	Midi	35mm	6mm	29mm
X1U1R126232	10	Midi	35mm	6mm	32mm
X1U1R126235	10	Midi	35mm	6mm	35mm
X1U1R226213	10	Midi	35mm	6mm	13mm Starter Hole
X1U1R226222	10	Midi	35mm	6mm	22mm
X1U1R226225	10	Midi	35mm	6mm	25mm
X1U1R226229	10	Midi	35mm	6mm	29mm
X1U1R226232	10	Midi	35mm	6mm	32mm
X1U1R226235	10	Midi	35mm	6mm	35mm
X1U1R326213	10	Midi	35mm	6mm	13mm Starter Hole

Aurum® Plus Soft Convex Urostomy

Product Code	Pack Size	Bag Size	Tension Location	Depth	Aperture Size
X1U1R136213	10	Maxi	35mm	6mm	13mm Starter Hole
X1U1R136222	10	Maxi	35mm	6mm	22mm
X1U1R136225	10	Maxi	35mm	6mm	25mm
X1U1R136229	10	Maxi	35mm	6mm	29mm
X1U1R136232	10	Maxi	35mm	6mm	32mm
X1U1R136235	10	Maxi	35mm	6mm	35mm
X1U1R236213	10	Maxi	35mm	6mm	13mm Starter Hole
X1U1R236222	10	Maxi	35mm	6mm	22mm
X1U1R236225	10	Maxi	35mm	6mm	25mm
X1U1R236229	10	Maxi	35mm	6mm	29mm
X1U1R236232	10	Maxi	35mm	6mm	32mm
X1U1R236235	10	Maxi	35mm	6mm	35mm
X1U1R336213	10	Maxi	35mm	6mm	13mm Starter Hole
X1U1R126313	10	Midi	48mm	6mm	13mm Starter Hole
X1U1R126332	10	Midi	48mm	6mm	32mm
X1U1R126335	10	Midi	48mm	6mm	35mm
X1U1R226313	10	Midi	48mm	6mm	13mm Starter Hole
X1U1R226332	10	Midi	48mm	6mm	32mm
X1U1R226335	10	Midi	48mm	6mm	35mm
X1U1R326313	10	Midi	48mm	6mm	13mm Starter Hole
X1U1R136313	10	Maxi	48mm	6mm	13mm Starter Hole
X1U1R136332	10	Maxi	48mm	6mm	32mm
X1U1R136335	10	Maxi	48mm	6mm	35mm
X1U1R236313	10	Maxi	48mm	6mm	13mm Starter Hole
X1U1R236332	10	Maxi	48mm	6mm	32mm
X1U1R236335	10	Maxi	48mm	6mm	35mm
X1U1R336313	10	Maxi	48mm	6mm	13mm Starter Hole



Aurum® Plus Flat Closed




Product Code	Pack Size	Bag Size	Aperture Size
X1C1A11S	30	Mini	13mm starter hole (cut to 50mm x 70mm)
X1C1A21S	30	Mini	13mm starter hole (cut to 50mm x 70mm)
X1C1A12S	30	Midi	13mm starter hole (cut to 60mm x 80mm)
X1C1A1220	30	Midi	20mm
X1C1A1225	30	Midi	25mm
X1C1A1229	30	Midi	29mm
X1C1A1232	30	Midi	32mm
X1C1A1235	30	Midi	35mm
X1C1A1238	30	Midi	38mm
X1C1A1244	30	Midi	44mm
X1C1A22S	30	Midi	13mm starter hole (cut to 60mm x 80mm)
X1C1A2220	30	Midi	20mm
X1C1A2225	30	Midi	25mm
X1C1A2229	30	Midi	29mm
X1C1A2232	30	Midi	32mm
X1C1A2235	30	Midi	35mm
X1C1A2238	30	Midi	38mm
X1C1A2244	30	Midi	44mm

Aurum® Plus Flat Closed

Product Code	Pack Size	Bag Size	Aperture Size
X1C1A32S	30	Midi	13mm starter hole (cut to 60mm x 80mm)
X1C1A13S	30	Maxi	13mm starter hole (cut to 60mm x 80mm)
X1C1A1320	30	Maxi	20mm
X1C1A1325	30	Maxi	25mm
X1C1A1329	30	Maxi	29mm
X1C1A1332	30	Maxi	32mm
X1C1A1335	30	Maxi	35mm
X1C1A1338	30	Maxi	38mm
X1C1A1344	30	Maxi	44mm
X1C1A23S	30	Maxi	13mm starter hole (cut to 60mm x 80mm)
X1C1A2320	30	Maxi	20mm
X1C1A2325	30	Maxi	25mm
X1C1A2329	30	Maxi	29mm
X1C1A2332	30	Maxi	32mm
X1C1A2335	30	Maxi	35mm
X1C1A2338	30	Maxi	38mm
X1C1A2344	30	Maxi	44mm
X1C1A33S	30	Maxi	13mm starter hole (cut to 60mm x 80mm)



KEY

-  Black colour bag
-  Sand colour bag
-  Clear bag



Aurum® Plus Flat Drainable

Product Code	Pack Size	Bag Size	Aperture Size
X1D1A11S	30	Mini	13mm starter hole (cut to 50mm x 70mm)
X1D1A21S	30	Mini	13mm starter hole (cut to 50mm x 70mm)
X1D1A12S	30	Midi	13mm starter hole (cut to 60mm x 80mm)
X1D1A1220	30	Midi	20mm
X1D1A1225	30	Midi	25mm
X1D1A1229	30	Midi	29mm
X1D1A1232	30	Midi	32mm
X1D1A1235	30	Midi	35mm
X1D1A1238	30	Midi	38mm
X1D1A1244	30	Midi	44mm
X1D1A22S	30	Midi	13mm starter hole (cut to 60mm x 80mm)
X1D1A2220	30	Midi	20mm
X1D1A2225	30	Midi	25mm
X1D1A2229	30	Midi	29mm
X1D1A2232	30	Midi	32mm
X1D1A2235	30	Midi	35mm
X1D1A2238	30	Midi	38mm
X1D1A2244	30	Midi	44mm

Aurum® Plus Flat Drainable

Product Code	Pack Size	Bag Size	Aperture Size
X1D1A32S	30	Midi	13mm starter hole (cut to 60mm x 80mm)
X1D1A13S	30	Maxi	13mm starter hole (cut to 60mm x 80mm)
X1D1A1320	30	Maxi	20mm
X1D1A1325	30	Maxi	25mm
X1D1A1329	30	Maxi	29mm
X1D1A1332	30	Maxi	32mm
X1D1A1335	30	Maxi	35mm
X1D1A1338	30	Maxi	38mm
X1D1A1344	30	Maxi	44mm
X1D1A23S	30	Maxi	13mm starter hole (cut to 60mm x 80mm)
X1D1A2320	30	Maxi	20mm
X1D1A2325	30	Maxi	25mm
X1D1A2329	30	Maxi	29mm
X1D1A2332	30	Maxi	32mm
X1D1A2335	30	Maxi	35mm
X1D1A2338	30	Maxi	38mm
X1D1A2344	30	Maxi	44mm
X1D1A33S	30	Maxi	13mm starter hole (cut to 60mm x 80mm)



Aurum® Plus Flat Urostomy

Product Code	Pack Size	Bag Size	Aperture Size
X1U1E12S	10	Midi	13mm starter hole (cut to 55mm)
X1U1E22S	10	Midi	13mm starter hole (cut to 55mm)
X1U1E32S	10	Midi	13mm starter hole (cut to 55mm)
X1U1E13S	10	Maxi	13mm starter hole (cut to 55mm)
X1U1E1320	10	Maxi	20mm
X1U1E1325	10	Maxi	25mm
X1U1E1329	10	Maxi	29mm
X1U1E1332	10	Maxi	32mm
X1U1E1335	10	Maxi	35mm
X1U1E1338	10	Maxi	38mm
X1U1E23S	10	Maxi	13mm starter hole (cut to 55mm)
X1U1E2320	10	Maxi	20mm
X1U1E2325	10	Maxi	25mm
X1U1E2329	10	Maxi	29mm
X1U1E2332	10	Maxi	32mm
X1U1E2335	10	Maxi	35mm
X1U1E2338	10	Maxi	38mm
X1U1E33S	10	Maxi	13mm starter hole (cut to 55mm)



Order your samples at
www.wellandmedical.com/aurumplus



Welland Medical Ltd., Hydehurst Lane, Crawley, West Sussex RH10 9AS United Kingdom

Tel: +44 (0) 1293 615455 **Fax:** +44 (0) 1293 615411 **Email:** info@wellandmedical.com

©Welland Medical 2025 • Welland Medical Ltd. is a CliniMed® Group Company • Welland®, the Welland logo™ and Aurum® are trademarks of CliniMed (Holdings) Ltd.

www.wellandmedical.com

PID 11874 V2